


dermaJet

*State of the art technology at the
service of your patients*

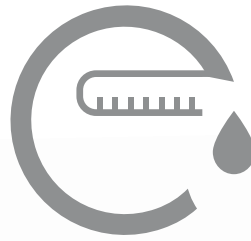

**Advanced - Precise
Convenient - Safe ...**



For application of injected active ingredients



Indispensable for optimising application
Techniques such as:

- PRP -
- BOTULINUM TOXIN -
- HYALURONIC ACID -
- FILLERS -
- MESOTHERAPY -
- HYPERHIDROSIS -
- VASCULAR -
- BLANCHING -



Allows application of microdoses of up to 0.001 ml.

Its electronic system permits
application of high viscosity
ingredients **using
thinner needles.**





Application of PRP with Dermajet and Paskin 34G triple-tip needles.

DERMAJET + PASKIN

DermaJet **facilitates the application of high density** ingredients with Paskin 34G triple-tip needles.

The needles distribute the dose across three entry points **minimising papule size**, resulting in less trauma and discomfort for the patient.

Recovery time is reduced to the minimum

APPLICATION OF FILLERS

DermaJet allows you to work in **continuous mode**, facilitating **continuous retrograde** filler application for filling wrinkles and folds or creating volume, using thinner needles.

Application using DermaJet provides **uniform pressure, ensuring homogeneous results**. If the product is applied with constant pressure and speed, the patient's sensation of discomfort decreases. Additionally, the use of thinner needles minimises discomfort.

It permits the application of controlled doses of hyaluronic acid for filling fine lines, treating rings under the eyes or areas requiring the utmost precision.





DermaJet is an advanced injection device that can deliver **precise doses**, adapted to each treatment, ensuring optimum results and product usage.



Application of high viscosity non-reticulated hyaluronic acid (3.5%) with a 34G microneedle (PASNY)



Application of high viscosity non-reticulated hyaluronic acid (3.5%) with a 34G triple-tip microneedle (PASKIN)

MICRODOSING

Allows application of microdoses of up to 0.001 ml facilitating treatments that require the utmost accuracy, such as filling fine superficial lines, applying toxin with minimal dilution or the treatment of fine superficial veins.

BLANCHING

Hyaluronic acid blanching is used to treat superficial wrinkles such as "bar code" lines, fine décolletage/chest lines or fine facial lines on the cheeks, forehead, etc.

DermaJet is ideal for minimally invasive and painless treatments of this type as it allows application of highly accurate microdoses.

THINNER NEEDLES

With DermaJet there is no need to apply any pressure to the plunger of the syringe. DermaJet does it automatically and precisely. Not only does this ensure the correct dose on each occasion, **but it also allows the use of much thinner needles**, which would be impossible with manual application.

Patient pain and discomfort is reduced and recovery time is minimised.





Application of PRP with Dermajet and Pasny 34G needles.

SAFER AND FASTER TREATMENTS

Lengthy procedures, which require application of a large number of doses (PRP, stem cells, etc) can be carried out much **more comfortably and safely**, with no risk that manual application might result in differing dosages. The dose applied will always be **accurate and appropriate**, resulting in better product usage and less discomfort for the patient.

Dermajet can be used with 1ml or 5ml syringes, adapting to a wide range of treatments. Supports most preloaded syringes on the market. Allows the plunger to be drawn back in order to guarantee needle position.

APPLICATIONS

Suitable for a wide range of uses:

- PRP
- Botulinum toxin
- Hyaluronic acid, fillers
- Mesotherapy
- Hyperhidrosis
- Vascular sclerotherapy
- Blanching

ADVANTAGES


- **State of the art technology at the service of your patients**
- Minimal margin of error for the professional
- Cordless
- More comfortable procedures
- Fast and accurate
- Avoids fatigue and prevents wounds





DERMICA LABORATOIRES AG
Reitergasse n°1, 2nd floor.
8004. Zurich. Switzerland.
info@dermica.ch

www.dermica.com

Follow us on 

DERMICA LABORATOIRES EUROPE S.L
Avd. Ciclista Mariano Rojas 76.
1ª planta. 30009. Murcia. Spain.
info@dermica.ch
T: +34968416972