

FirmZon

Specially indicated for the treatment of cellulite and localized fat.

Micro-needling Therapy with Goldpen + Firmzon



Firmzon has an incredible ability to metabolize and produce energy. It eliminates fatty acids, facilitating their removal via the lymphatic system. Results are a visible reduction in volume in the treated areas.

Facial and body liposculpture is a technique used to recontour the skin using mesotherapy. This facilitates the treatment of unsightly fat deposits in parts of the body that do not respond to dieting. This method provides a gentle recontouring and a volume reduction on the face and body, achieving a more harmonious appearance.

By applying the solution with **Goldpen micro-needling therapy**, results are improved. The action of the micro-needles enhances collagen and elastin production, improving the appearance of skin. Since it improves the synthesis of collagen and elastin, it will help the skin to recover and tighten as the presence of fat decreases.

INDICATIONS

This method provides a gentle recontouring and a volume reduction on the face and body, achieving a more harmonious appearance. Especially recommended for the treatment of:

- Palpebral Bags
- Jowl and cheeks

For bigger areas, we recommend mixing Firmzon ampoules with L-Carnitine ampoules in the same session to achieve better results.

PROTOCOL

- Step 1 If necessary, apply a desensitizing cream on the area to be treated to prevent discomfort for the patient during the procedure. Allow it to act for half an hour.
- Step 2 Remove remaining desensitizing cream and clean the skin with a soft tissue.

- Step 3 Apply a disinfectant solution on the area to be treated.
- Step 4 The patient should lie down face down for a more comfortable process.
- Step 5 Open one ampoule of Firmzon and load its contents into a syringe using a discharge needle (e.g. 18G)
- Step 6 Insert a new cartridge needle in your Goldpen and remove the cap.
- Step 7 Adjust the needle for desired depth (<0.5mm eye contours, 1.5 jowls and checks)
- Step 8 Apply a few drops of Firmzon solution on the area to be treated. Do not apply the product on the entire surface at once to prevent spillage while turning the patient's head during the procedure. It is better to apply a few drops as you advance with the Goldpen.
- Step 9 Keep a slight tension on to the skin to the surface flat and apply the Goldpen in the area by drawing small circles or lines on the skin. Continue for a few minutes, but remember, it is not necessary to provoke a strong erythema to achieve good results. The Goldpen micro-needles act at 120 cycles per second, so it might occur that redness appears slower than you proceed with the treatment. If after the treatment there are areas where skin stays white, without redness, you may need to repeat the procedure in this area.. It is possible to provoke some small micro-bleeding in the area. This is absolutely normal.
- Step 10 Once you finish the procedure with Firmzon on the area, apply a Hydra Gel Mask and leave it on for 20 minutes.

Repeat the procedure with the Goldpen and Firmzon once a week for at least 4-5 weeks. After, according to the doctor's criteria, it would be advisable to perform additional sessions at intervals of one month or one and a half months.

The action of the micro-needles will help to prevent loose and saggy skin while fat deposits are reduced. To help to tighten skin it is advisable to **mix Firmzon and Liftingzon** in the same session. In this way, at the same time Firmzon acts by diluting and draining the fat, Liftingzon will act by tightening and firming the skin.

AFTER TREATMENT

Avoid using make-up for the following 24 hours.

Do not visit a sauna, practise intense sports or apply any other facial treatments for the following 48 hours.

Use sunscreen daily for several days after micro-needling treatment and avoid sun exposure. In any case, we recommend the use of sunscreen daily as a normal habit.

RECOMMENDATIONS DURING PROCEDURE.

1. Make sure you select the right depth and speed of the needles for each area to be treated.
2. As a general rule, the deeper the needles go, the faster they should work. However, we always recommend using the fastest speed in order to avoid needles hooking and scratching the skin.
3. Needle Cartridges are for single use only. Do not re-use the same cartridge, even for the same patient.
4. Do not press your Goldpen on patient's skin - move it smoothly. If you need the needles to go deeper, adjust the depth on your Goldpen.

CLINICAL RESULTS

