

# JALOR

---

PURE HYALURONIC ACID  
DERIVED FROM BIO-FERMENTATION  
THUS 100% NON ANIMAL SOURCED

---

---



---

Dermica Laboratoires is pleased to present its line of products based on Hyaluronic Acid that guarantee doctors and patients the utmost convenience, satisfaction and reliability during both application and their biostimulating action.



Facial modeling • Wrinkle correction • Skin rejuvenation

---

## ADVANTAGES AND DISTINGUISHING FEATURES

The treatment is immediate, since it requires only a few minutes and can be carried out straight after a simple medical consultation.

The application is virtually painless and generally requires no anaesthesia of any sort.

The glass JALOR syringes are very easy to handle, practic for application, allow considerable control over the amount injected and give an excellent finished filling effect.

The results of JALOR tratment are long-lasting but not permanent; indeed, over time natural reabsorption of the pure hyaluronic acid occurs.

Depending on the type of treatment, with the aim of maintaining a youthful and natural appearance, the patient tends to repeat the application with JALOR after a few months.



## PURIFICATION AND HOMOGENISATION.

The entire JALOR range benefits from a very advanced purification and homogenisation process:

- After having aligned the hyaluronan chains, the next step is cross-linking with BDDE, thus creating, using the latest innovations, a three-dimensional matrix with a very fine mesh, making it possible to obtain a more homogeneous gel than other formulas.
- Immediately after this, a new, exclusive stage of homogenisation and optimisation of the gel's visco-elasticity is used to make the injection fluid and regular, so the product will be highly viscous and elastic.
- All this makes it possible for a very dense gel to pass through a 27G needle, with spectacular effects and exceptional duration.



## re-Style



### BIO-STIMULATION AT THE HIGHEST LEVEL.

The particular formulation of JALOR re-Style, with its extremely concentration of hyaluronic acid ( 18 mg/1 ml ) together with a high concentration of glycerol ( 20 mg/ml ) and ultra high molecular weight ( 3 million Daltons: double or even three times that of many rival formulas ), allows the reaction of the optimum physiological environment for fibroblast proliferation and migration and for the neo-synthesis of collagen, elastin and other constituents of the extracellular matrix.

Jalor Re-Style is suitable for all areas of the face and is also ideal for the treatment of the neck, décolletage/chest area and the backs of the hands, when following anti-ageing programmes recommended by your specialist. An initial course of two or three treatments at close intervals followed by follow-on sessions every few months is recommended.

Content	Concentration	Reticulating agent	Molecular weight	Needles
1 ml	HA 18mg/ml	Glycerol 20mg/ml	3m Daltons	2 x 27G

## Style



### THE FILLER FOR EVERY SOLUTION.

The latest research on the physical chemistry of reticulation has allowed the development of a brilliant product that provides a unique solution to the wants of doctors and patients alike. A natural, safe, convenient and easy-to-use product that provides long-lasting results. Jalor Style is the ideal multipurpose filler for all manner of cosmetic treatments.

- Non-animal origin: Sodium hyaluronate synthesised by bio-fermentation.
- Dual action, high-concentration filler: 23 mg/1 ml of cross-linked hyaluronic acid.
- Ultra high molecular weight product ( 3 million Daltons ): this characteristic, together with cross-linkage, allows us to increase the resistance to degradation by hyaluronidases to a maximum and thus maximise the duration of the product in the tissue.
- Cross-linking with butanediol diglycidyl ether (BBDE), the cross-linking agent universally recognised as the least toxic of all, which ensures that the risk of allergies is minimal.

Content	Concentration	Reticulating agent	Molecular weight	Degree of Crosslinking	Dinamic viscosity	Needles
1 ml	HA 23mg/ml	BBDE	3m Daltons	***	500.000 Pas	2 x 27G




Style re-Style intense SweetTouch SweetKiss

TREATMENT

1. Forehead wrinkles	— — —	—
2. Periocular wrinkles	— — —	—
3. Cheeks	— —	
4. Naso genial folds	— — — —	—
5. Perilabial wrinkles	— — —	—
6. Labial commissures	— — —	—
7. Increase in lip volume	—	—
8. Lip contour	—	—
9. Facial wrinkles	— — —	—
10. Nasal correction		— —
11. Glabella	— — —	—
12. Neck	—	—
13. Décolleté	—	
14. Hand	—	
15. Chin		—



## intense

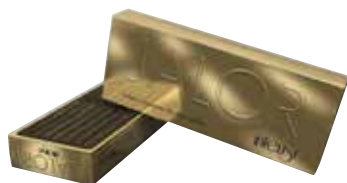

**FOR ALL TYPES OF VOLUME.**

The excellent results achieved with JALOR Style led to the creation of a specific product that gives volume to the face while maintaining the characteristics of the JALOR range: the result is a highly cross-linked gel (also containing 23 mg/1 ml of hyaluronic acid) which is injected with a 27G needle to reach the desired level of filling without causing trauma to the tissues.

Jalor Intense fills lines and treats deep wrinkles in thick skin. Restores or creates facial volume, increases the volume of the lips.

The duration is exceptional for a reabsorbable product: it varies between 8 and 18 months following implantation.

Content	Concentration	Reticulating agent	Dynamic viscosity	Degree of Crosslinking	Needles
1 ml	HA 23mg/ml	BBDE	500.000 Pas	****	2 x 27G

## Sweet Kiss



**HYALURONIC ACID CROSS-LINKED WITH LIDOCAINE.**

Rehydrates and illuminates, eliminates deep wrinkles, recovers volumes and facial contours, without changing facial features or losing expressiveness.

Content	Concentration	Reticulating agent	Dynamic viscosity	Degree of Crosslinking	Needles	Lidocaine
1 ml	HA 23mg/ml	BBDE	1.950.000 million Pa.s	****	2 x 27G	0,3%

## Sweet Touch



**HYALURONIC ACID CROSS-LINKED WITH LIDOCAINE.**

Valid for any aesthetic correction, it is integrated into your skin and accompanies your expressive and natural gestures.


Content	Concentration	Reticulating agent	Dynamic viscosity	Degree of Crosslinking	Needles	Lidocaine
1 ml	HA 23mg/ml	BBDE	450.000 million Pa.s	***	2 x 27G	0,3%





DERMICA LABORATOIRES AG  
Reitergasse n°1, 2nd floor.  
8004. Zurich. Switzerland.  
info@dermica.com

[www.dermica.com](http://www.dermica.com)

Follow us on 

DERMICA LABORATOIRES EUROPE S.L  
Avd. Ciclista Mariano Rojas 76  
1ª planta. 30009. Murcia SPAIN  
info@dermica.es  
T: +34968416972