

LiftingZon

For firmer, smoother and more hydrated skin with fewer wrinkles

Micro-needling Therapy with Goldpen + Liftingzon



LiftingZon has an immediate firming effect on the skin, restoring fibroblasts and stimulating collagen production. It acts as an antioxidant and stabiliser of the cell membrane, protecting it from damage caused by free radicals. It promotes the regeneration of fibroblasts and aids in skin recovery.

By applying the solution with Goldpen micro-needling therapy, results are improved. The action of the micro-needles enhance production of collagen and elastin and has a skin renovating effect. The biological actives stay in the dermis longer than if applied by injecting through mesotherapy, achieving better results in a shorter period of time.

INDICATIONS

LiftingZon is indicated for the treatment of **loose skin and saggy areas**. Its main components are DMAE and Organic Silicon, which have an immediate effect, but it is through a programme of ongoing sessions that long-term results will become apparent.

PROTOCOL

It is advisable to begin the treatment by applying a peeling gel (**Step 1 peeling system**) at least one week before the first session with Liftingzon and Goldpen to enhance results. The skin will be cleaner and more ready to react to the actives in the applied solution, and at the same time, it will have already been induced to create new collagen and elastin.

- Step 1 If necessary, apply a desensitizing cream on the area to be treated to prevent discomfort for the patient during the procedure. Allow it to act for half an hour.

- Step 2 Remove the desensitizing cream and clean the skin with a soft tissue.
- Step 3 Apply a disinfectant solution on the area to be treated.
- Step 4 The patient should remain laying down to facilitate comfort throughout the process.
- Step 5 Open one ampoule of Liftingzon and load its contents into a syringe using a discharge needle (e.g. 18G)
- Step 6 Insert a new cartridge needle in your Goldpen and remove the cap.
- Step 7 Adjust the needle for desired depth (0.5mm forehead, <0.5 surrounding the eyes, 1,00 on the lips and neck, 1.5mm for the cheek)
- Step 8 Apply a few drops of Liftingzon solution on the area to be treated. Do not apply the product on the entire surface to prevent spillage while turning the patient's head during the procedure. It is better to apply a few drops as you advance with the Goldpen.
- Step 9 Pull slightly to maintain the surface of the skin flat and apply the Goldpen on the area by drawing small circles or lines on the skin. Continue for a few minutes, but remember, it is not necessary to provoke a strong erythema to achieve good results. The Goldpen micro-needles act at 120 cycles per second, so it might occur that redness appears slower than you proceed with the treatment. If after the treatment there are areas where skin stays white, without redness, you might need to repeat the procedure in this area. Micro-bleeding may occur in treated areas. This is perfectly normal.
- Step 10 Once you complete the procedure with Liftingzon Solution on the area, apply a calming cream (Comfort Cream) for body treatments or a Hydra Gel Mask for face treatment and leave it on for 20 minutes.

Repeat the procedure with the Goldpen and Liftingzon once a week for at least 4-5 weeks. After and following the doctor's indications more sessions may be advisable at one or one and a half month intervals. Repeat a maintenance treatment after one year.

AFTER TREATMENT

Avoid using make-up during the following 24 hours.

Do not visit a sauna, practise intense sports or apply any other facial treatments for the following 48 hours.

Always use sunscreen for a few days after a micro-needling treatment and avoid sun exposure. In any case, we recommend the use of a sunscreen cream as a daily habit.

RECOMMENDATIONS DURING THE PROCEDURE.

1. Make sure you select the right depth and speed of the needles for each area to be treated.

2. As a general rule, the deeper the needles go, the faster they should work. However, we always recommend using a faster speed in order to avoid needles from hooking and scratching the skin.
3. Needle Cartridges are single use only. Do not re-use the same cartridge, even for the same patient.
4. Do not press your Goldpen on the patient's skin - move it smoothly. If you need the needles to go deeper, adjust the depth on your Goldpen.

CLINICAL RESULTS



Before



After 4 sessions