


GoldPen

Auto micro-needle system

Therapies For
Skin Regeneration

MICRONEEDLE THERAPIES

GoldPen is the safest, most hygienic, fastest and most efficient transdermal delivery system. Its unique microneedling system creates thousands of microchannels quickly and safely, increasing collagen and elastin production in a safe and natural way for the patient. GoldPen is a practical and effective tool to help maximise the impact of anti-ageing facial and body treatments. It ensures active ingredients are delivered in the correct location and in the right way.

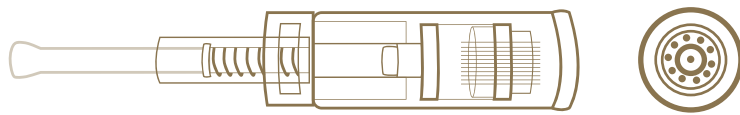

It has a single-use head with **11 microneedles**. Depth of penetration can be regulated **between 0mm and 2.5mm**. Needle speed up to 120 cycles / second. It creates up to **1,200 microchannels per second**, hence greater efficiency in the delivery of active ingredients.




ADVANTAGES

- **Less redness and scleroderma.** - Eliminates side effects, keeping skin softer.
- **Shorter recovery time** after each session.
- **Easier handling.** Simple and easy to transport. Low cost supplies.
- **Extremely versatile** for a wide range of treatments. Alopecia, wrinkles, age spots, scars, stretch marks, PRP, melasma, hyperhidrosis, mesobotox...
- The use of **disposable needles** eliminates risk of infection.
- Treatments can be carried out **faster and with greater precision** than with traditional rollers.
- Needles can be adjusted to a depth ranging from **0 to 2.5 mm**.
- Much **less pain** and shorter treatment time.


GoldPen

- Stimulates follicle activation in hair loss treatments.
- Brighter-looking skin. Improves appearance and texture of skin.
- Helps eliminate age spots, exfoliating the outer layers and deeply regenerating the skin.
- Activates and stimulates natural collagen production.
- Reduces expression lines.
- Visible improvement to stretch marks, scars and keloids.
- Other applications: PRP, mesobotox, alopecia, melasma treatments, hyperhidrosis treatments, etc...
- Up to 90% penetration of active ingredients, increasing their effectiveness.

GoldPen

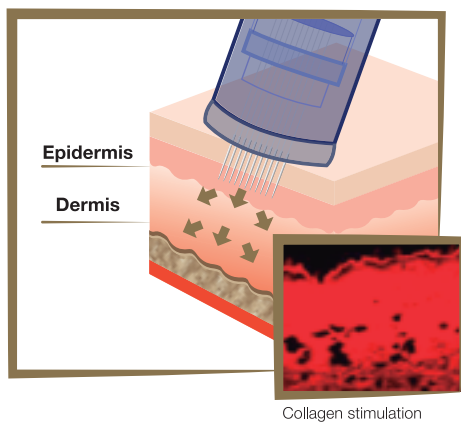

Traditional roller

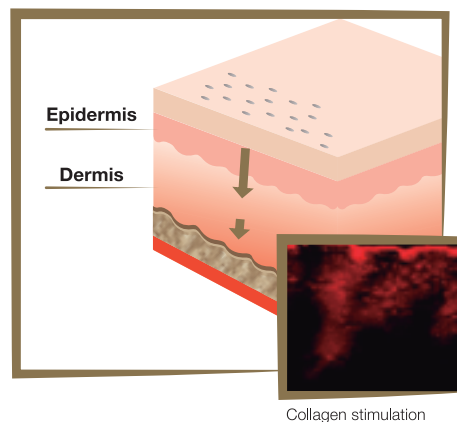

Enhanced penetration of active ingredients: **More EFFECTIVE**

THE BEST COMBINATION


GoldPen & Anti-Ageing Solutions




All our solutions could be mixed for enhance the effectivity of the treatment.

GUIDLINES FOR USE

	CYCLES / SEC.	MICROCHANNELS / SEC.
LEVEL 1	20-25 cycles	220-275
LEVEL 2	30-35 cycles	330-385
LEVEL 3	40-45 cycles	440-495
LEVEL 4	50-55 cycles	550-605
LEVEL 5	60-65 cycles	660-751
LEVEL 6	70-75 cycles	770-825
LEVEL 7	80-85 cycles	880-935
LEVEL 8	115-120 cycles	1145-1200

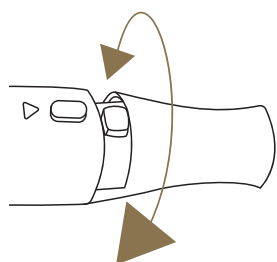

Speed Selection

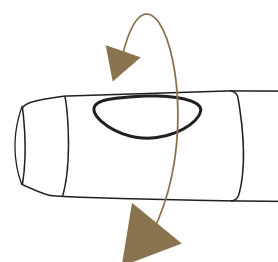

Depth Selection

○
Forehead and brow
Fine skin
 0.5 - 0.75 mm
Recommended speed
 2-3
Thick skin
 0.75 - 1.00 mm
Recommended speed
 4-5

○
Nose
Fine skin
 0.75 - 1.00 mm
Recommended speed
 2-3
Thick skin
 1.00 mm
Recommended speed
 4-5

○
Lips
Fine skin
 0.25 mm
Recommended speed
 1-2
Thick skin
 0.25 mm
Recommended speed
 1-2

○
Eye area
Fine skin
 0.25 mm
Recommended speed
 2-3
Thick skin
 0.25 mm
Recommended speed
 2-3

○
Cheeks
Fine skin
 1.00 mm
Recommended speed
 4-5
Thick skin
 1.00 - 1.25 mm
Recommended speed
 4-5

○
Scars
Fine skin
 1.25 - 1.5 mm
Recommended speed
 5-6
Thick skin
 1.25-1.5 mm
Recommended speed
 5-6

○
Stretch marks
Fine skin
 1.5 - 2.0 mm
Recommended speed
 6-7
Thick skin
 1.5-2.5 mm
Recommended speed
 6-7



As a general rule, the greater the needle depth, the greater the speed required.

dermica[®]
laboratoires

DERMICA LABORATOIRES AG
Reitergasse n°1, 2nd floor.
8004. Zurich. Switzerland.
info@dermica.ch

www.dermica.ch

Follow us on 

DERMICA LABORATOIRES EUROPE S.L
Avd. Ciclista Mariano Rojas76.
1ª planta. 30009. Murcia. Spain.
info@dermica.ch
T-968416972